

CARING FOR THE LOCK 2000 VEST



Cleaning

- ▶ Clean the Vest no less than once a week AND when soiled.*
- ▶ Before washing, engage all velcro, including the Single and Dual Safe Strips.
- ▶ Machine wash. Gentle cycle, cold water. Hang dry only (no dryer).
- ▶ Use standard laundry detergent.
- ▶ Vest may be hand-washed using a clean basin if a washer is not available.
- ▶ Cannot be cleaned in commercial laundry.

**Individual hospital and patient population protocols may vary.*

Inspection & Maintenance:

- ▶ Examine the Vest to determine that all parts are present and in working order.
- ▶ Do not use if Vest is damaged or missing parts.
- ▶ The LOCK 2000 Vest can be reused for the same patient after cleaning.



For More Information
Clinical.GusGear.com



Contact Us
866-822-4487 (4GUS)



Our Location
PO Box 607, Valencia, PA 16059



Email Us
Info@GusGear.com