



Line Cover Instructions

OPEN

Open the Velcro on the entire length of the cover. Place your lines inside the cover.



CLOSE


Close the Velcro on the entire length of the cover.



WRAP

Wrap line(s) tightly on each end with Velcro tab to prevent the cover from shifting on the line(s).



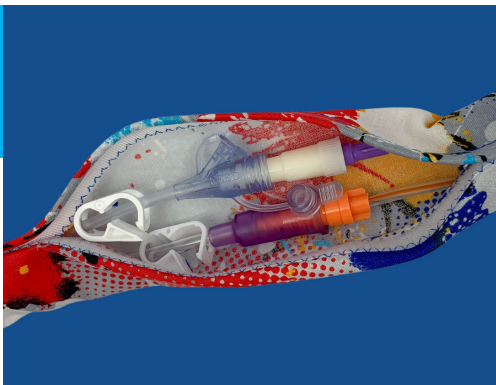

ACCESS

To easily access the line(s), simply open a small portion of the Velcro where your connections or med port are located.

CARE INSTRUCTIONS

- ✓ Fasten all Velcro before washing
- ✓ Machine-wash cold, gentle cycle
- ✓ Open and reshape, then lay flat or hang to dry
- ✓ Touch up with iron, as needed — do not iron Velcro



SCAN TO WATCH
A HOW-TO VIDEO



Visit GusGear.net to
see our full product
line or write reviews.

GUS & GEAR
BREAK FREE!

© 2021 Gus Gear LLC. Patent
pending. All rights reserved.