



Ostomy Pouch Cover Instructions

REACH

Reach your hand through the front opening and out the hole in the back.



INSERT

Insert pouch into cover through back opening.



OPTIONAL

For ostomy belt use:
Feed plastic belt loops on pouch through access slots on back of pouch. Hook ostomy belt onto each tab.



TUCK

Tuck in tail of pouch, and smooth front opening to flatten. For urostomy or high-output pouches, fold spout up as needed to fit.

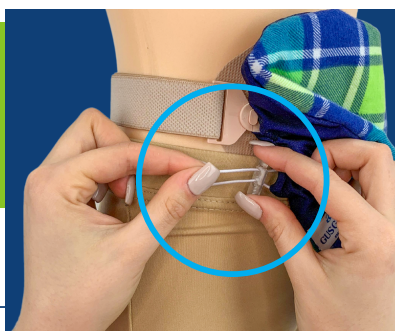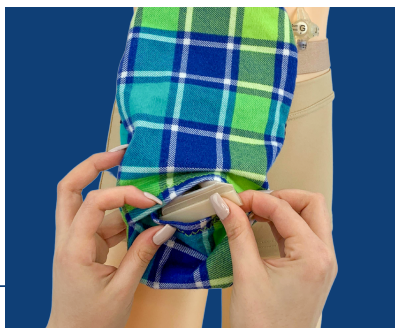

TIGHTEN

Tighten cord stop to secure around flange —
DO NOT OVER-TIGHTEN.

EMPTY

To empty, pull tail out front opening and push cover up as needed. **THERE IS NO NEED TO REMOVE COVER.**



CARE INSTRUCTIONS

- ✓ Machine-wash cold, gentle cycle.
- ✓ Reshape, then lay flat or hang to dry.
- ✓ Touch up with iron, as needed.
- ✓ **DO NOT IRON STRETCH FABRIC AROUND BACK OPENING.**

SCAN TO
WATCH A
HOW-TO
VIDEO



Visit GusGear.net to see our full product line or write reviews.

GUSGEAR™
BREAK FREE!

© 2021 Gus Gear LLC. Patent pending. All rights reserved.