



IFU – Instructions for Use

Description	Yewtwist is a unique tool designed to quickly, safely and efficiently disconnect most IV and feeding tube connections. It has three distinct jaw sizes to accommodate a wide range of IV and feeding tube connections. The jaws provide grip and torque without damaging the IV line or feeding tube. It was designed to provide additional grip strength.
Indication for use	Yewtwist is intended to be used to assist a user with disconnection of luer lock fittings, including fittings used for medical purposes, by providing additional grip and leverage to improve the user's ability to manipulate and disconnect an overtightened and difficult to disconnect or adhered fitting. * This tool is intended for single patient- multiple use.
Directions for use	1. Select jaw size and place on connection furthest from patient. 2. Pinch arms of Yewtwist and turn counter clock-wise. 3. Detach.
Mechanism of action	Instructions for use/Mode d'emploi  1. Select jaw size 2. Pinch Yewtwist™ and turn counter clock-wise 3. Detach
Cleaning Instructions	Clean Yewtwist with hospital grade wipes before each use. Yewtwist is nonsterile.
Warnings and precautions	Not made with natural rubber latex. Pinch point hazard. Not a toy – keep out of reach of small children and pets. This product is nonsterile. Do not pull on IV line or feeding tube while Yewtwist is attached. No known contraindications. For hospital and home use. Store at room temperature.