

Cleaning Recommendations

Cleaning Guidelines for Safe Reuse



Routine Cleaning

- Clean Yewtwist before and after each use with hospital-grade disinfectant wipes.
- For single-patient use only



Cleaning Method

- 1 Wipe all surfaces thoroughly
- 2 Follow contact time
- 3 Allow to air dry



Tested & Compatible Disinfectants



Diversey™ Oxivir® Tb RTU Disinfectant Cleaner

Clorox Healthcare® Bleach Germicidal Cleaner

Clorox Healthcare® Hydrogen Peroxide Cleaner

Super Sani-Cloth® Wipes

Metrex CaviWipes™



- ✓ No material degradation observed after **100 cleaning cycles**
- ✓ Follow disinfectant manufacturer's
- ✓ Safe for all Yewtwist surfaces



Important Information

- ⚠ Clean Wipe only
- ⚠ Do not soak or submerge
- ⚠ May use any hospital-grade wipes



Sterilization Notice:

Yewtwist has NOT been tested for autoclaving or sterilization.

- Steam
- Ethylene oxide (EtO)
- Plasma or gas sterilization

Use of untested sterilization methods may result in material damage or reduces product lifespan.